

SHORTCUT SHEET

ENKIDU TABLET IMPACT



The Tablet Impact is the largest of the Enkidu Impact Communication devices. It uses digitized and synthesized speech. The Tablet combines a keyboard with a large touch screen to provide additional methods of message production. With a full sized screen, the Tablet offers even more space for button, making it ideal for phrase and symbol based communication pages. The device comes installed with 10 standard configurations.

TO USE

TO CREATE A CONFIGURATION

A configuration is a set of vocabulary pages. Typically, all of an individual's pages will be in one configuration.

Create a new configuration using a Standard Configuration as a starting point:

1. Go to **File Menu**.
2. Choose **Standard Configuration**.
3. Choose the configuration you would like to start with.
4. To Save your new configuration:
 - a. Go to **File Menu**
 - b. Choose **Configuration**
 - c. Choose **Save Configuration As**
 - d. Give your configuration a name

Create a new blank configuration:

1. Go to **File Menu**
2. Choose **Configuration**
3. Choose **New Configuration**
4. Type a name for the page
5. Choose the **Arrow** next to **Page Type**
6. Choose the Page Layout that you prefer for this page

TO CREATE A BUTTON THAT SPEAKS

1. Go to **View Menu**

2. Choose **Go To Edit Mode**
3. Choose the **Page To Modify**
4. Double click on **Button You Want To Speak**
5. Choose **Speak Label** from the Action Box
6. Type the label for the button
7. Choose **Browse** and then a **Picture**
8. Click **OK**
9. Go to **View Menu**
10. Choose **Go to Run Mode**

TO CREATE A BUTTON THAT SPEAKS MORE THAN ONE LABEL

1. Go to **View Menu**
2. Choose **Go To Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Speak Message** from the Action Box
6. Type the message for the button
7. Type a label for the button
8. Choose **Browse** and choose a **Picture**
9. Click **OK**
10. Go to **View Menu**
11. Choose **Go to Run Mode**

TO CHANGE A BUTTON'S PICTURE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Browse** and then choose a **new picture**
6. Click **OK**
7. Go to **View Menu**
8. Choose **Go to Run Mode**

TO CREATE A PAGE

Pages are levels of specific vocabulary linked together to create a vocabulary set specific to the device user.

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Go to **View Menu** again
4. Choose **Page Editing**
5. Choose **Add Page** (NOTE: There is also an ADD PAGE shortcut icon at the top of the screen when you are in Edit Mode. Choosing this icon will also create a new blank page).
6. Type a name for the page
7. Choose the **Arrow** next to **Page Type** and choose the **Page Layout** that you prefer.
8. Be sure to add a **Navigation button** to your page or you will not be able to navigate to or from this page.

TO ADD A NAVIGATION BUTTON TO A PAGE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Navigate** or **Visit** from the Action Box
6. (NOTE: Navigate will take you to a page and leave you there, while Visit will take you there for one selection and immediately return to the previous page).
7. Choose the **page** that you want to link to from the Navigate/Visit box
8. Type a **label** for the button
9. Choose **Browse** and choose a **Picture** that you want to represent your navigation button
10. Click **OK**
11. Go to **View Menu**
12. Choose **Go to Run Mode**

TO ADD A NAVIGATION BUTTON THAT ALSO SPEAKS

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Speak Message** from the Action Box
6. Type the message you want spoken into the Message Box
7. Choose **Navigate or Visit** from the Second Action Box
8. Choose the **page** that you want to link to from the Navigate/Visit Box
9. Type a **label** for the button
10. Choose **Browse** and choose a **picture** that you want to represent your navigation button
11. Tap **OK**
12. Go to **View Menu**
13. Choose **Go to Run Mode**

TO CREATE A BUTTON TO PERFORM A STANDARD FUNCTION

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Control** from the Action Box
6. Choose the **Function** that you want for the button to perform from the Function Box
7. Click **OK**
8. Go to **View Menu**
9. Choose **Go to Run Mode**

TO MODIFY BUTTON APPEARANCE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on a **Button**
 - a. To modify colors on a specific button, choose the This **button** tab and

- make your changes
- b. To modify colors, the font, or style on all buttons, choose the **All Buttons** tab and make your changes.
 - c. To modify the page background color, choose the **This Page** tab and choose the color of your choice
5. Click **OK**
 6. Go to **View Menu**
 7. Choose **Go to Run Mode**

TO MODIFY THE PRONUNCIATION OF A WORD

1. Go to **Tools**
2. Choose **Pronunciation Editor**
3. Choose **Add**
4. Type the word
5. Type the words proper pronunciation
6. Test by choosing **Try Pronunciation**
7. Choose **OK**

PRODUCT INFORMATION

Contact Information: Enkidu Research
2100 Wharton Street
Suite 400
Pittsburgh, PA 15203
(800) 344-1778
www.enkidu.net

Cost: \$4695

Created By: Debby McBride, AAC Connections, 04-04

Edited By: Tammy Sproule, South-Central BOCES, SWAAAC Coordinator, 04-01-04

For more SWAAAC Shortcut Sheets, go to www.swaaac.org, click on **AT Loan Bank** and then click on **Shortcut Sheets**.
Would you like a Shortcut Sheet that doesn't exist? Contact the WestAC office at (970) 248-0876 or carrie.foster@uchsc.edu.