



Shortcut Sheet

MT4, DV4 Directions

Turn on device with button on front left top
Turn off using same button to turn on.

To program:

Go under down arrow

Page editing

Page editor

Touch button so outline of entire button is blue

Click on scissors to **delete**

To move: highlight and click on toolbar button with left arrow, click and drag button to move then click checkmark.

To change size: highlight in blue, click on button with arrows on each side, click and drag on white square to resize. Click checkmark when done.

When done moving and resizing, click on down arrow and Exit page editor

To change what a button says: Click on green box, pick button to change—modify button comes up

- click on rectangle to change label, highlight and retype message, OK
- click on rectangle to change audio cue and change it to what you want it to say. OK.
- click on symbol rectangle, click on search rectangle and type in symbol, pick symbol, OK.
- click on behavior rectangle, choose speak label, add, OK

To find out what page you are on: down arrow, setup, page navigator, the page you are on will be the one highlighted.

To go to a page: down arrow, setup, page navigator, click on page you want to go to on right hand side, then click on go to page at bottom.

Volume: down arrow, setup, speech controls

Soft reset: should be done daily (per Dynavox) down arrow, then soft reset at bottom of choices. It takes about 5 minutes.