

## SHORTCUT SHEET

### STEP-BY-STEP COMMUNICATOR WITH LEVELS



The Step-by-Step with Levels Communicator is a small multiple message device that uses digitized speech. It has 75 seconds of recording time. The Step-by-Step with Levels is designed for pre-recording a series of message sequences. A level can be considered as a folder where you can store a category of messages or a series for particular activity or setting. A mounting base or a travel strap may be added depending upon the student's needs.

#### TO USE

##### Battery Installation

1. Remove the mounting base (if attached).
2. Remove the battery door and insert a 9 volt battery.
3. Replace the battery door and mounting base.

##### To Record

1. Turn on by moving the volume control to the right until you hear a click,
2. Press the **Repeat/Rec** button **twice** and release.
3. Press and hold the top surface of the Step-By-Step device and record your message with the speaker grill facing you.
4. Release the top surface briefly.
5. Press and hold the top surface and record your next message.
6. Continue this process until all messages have been recorded in that level.
7. If no other levels are to be recorded: Press the **Repeat/Rec** button twice to end recording. If you wish to continue recording on the next level continue to step #8.
8. Press the **Repeat/Rec** button **three times** to advance to the next level.
9. Continue same process as above.

### To Play the Messages

1. Press the top surface of the Step-By-Step device to play each message.
2. Messages will be played in the order they were recorded, in that level, each time the top surface is pressed.
3. Once the last message in the sequence is played, the sequence will start over again for that level.

### To Play the Messages in the Next Level

1. Press the **Repeat/Rec** button **three times**.
2. Repeat the sequence for playing messages.

## OTHER FUNCTIONS AND DIRECTIONS FOR USE

1. All messages in a level must be recorded at one time. It is not possible to re-record or change just one message in the sequence without erasing all of the messages.
2. Pressing the **Repeat/Record** button twice will either begin or end the recording process.
3. Pressing the **Repeat/Record** button three times will advance you to the next level in both the record and play modes. Be sure to pause two seconds between pressing the **Repeat/Record** button three times.

## PRODUCT INFORMATION

**Contact Information:** Abelnet, Inc.  
1081 Tenth Ave. SE  
Minneapolis, MN 55414-1312  
(800) 322-0956  
[www.abelnetinc.com](http://www.abelnetinc.com)

**Pricing:** \$150.00

*Created by: Tammy Sproule, South-Central BOCES SWAAAC Coordinator; 04-01-04*

For more SWAAAC Shortcut Sheets, go to [www.swaaac.org](http://www.swaaac.org), click on **AT Loan Bank** and then click on **Shortcut Sheets**.  
Would you like a Shortcut Sheet that doesn't exist? Contact the WestTAC office at (970) 248-0876 or [carrie.foster@uchsc.edu](mailto:carrie.foster@uchsc.edu).